
OHROPAX® Classic

EN 352-2:2020, EU 2016/425

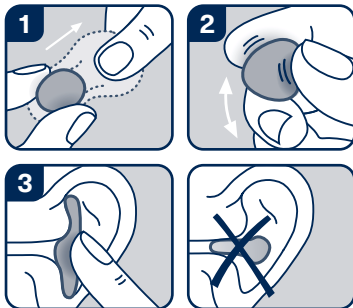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EN Wax earplugs against harmful noise for single use only

Instructions for use (see picture): 1. Before use, completely remove cotton wool. 2. Soften the plug by

kneading and shape it into a ball (no elongated form!). 3. Seal the ear canal tightly by gently pressing the ball. **Do not insert into the ear canal! Use entire earplug – do not split!**

Removal: Use your fingers to remove the plug in one piece. Do not use any auxiliary means! The plug can be loosened by slightly pushing the back of the ear.

Further information: • Insert the earplugs as described and check that they fit correctly. • Ensure hygienic use. Your hands, ears and earplugs should be clean and dry. • The product cannot be cleaned. • Wear earplugs at all times in noisy environments. • Failure to comply with these instructions may significantly impair the protective effect of the product. • This product may be adversely affected by certain chemical substances. Further information can be sought from the manufacturer. • Keep away from small children (danger of swallowing). • Can be used by children aged six and older; use only under adult supervision.

Ingredients: Vaseline, paraffin wax, cotton wool
Storage: In the box, dry and below 30° C.

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- A** CHEMOMEDICA GmbH, 1010 Wien, www.chemomedica.at
- P** Uriach Lusa S.A., 1495-061 Miraflôres www.uriach.com
OHROPAX GmbH